

AITECS PRO SP-12S SYRINGE PUMP FOR USE OF DOBUTAMINE (Myocardial)
UPDATED: MAY 2016

CPT CODE: N/A

MAXIMUM WEIGHT: 200 kg (pump dependent)

1. Make sure there is no air in syringe or line.
 2. Press **ON/OFF** button.
 3. Adjust syringe in syringe holder by using **BOLUS** and **>>** buttons. Close top clamp.
 4. Confirm type of syringe. **ENTER**.
 5. Dose mode should be $\mu\text{g}/\text{kg}/\text{min}$. If need to adjust, press **PROG**, then **mg \leftrightarrow mL** button, then **ENTER**.
 6. Concentration should be 5.00 mg/mL. If need to adjust, press **PROG**, then **mg \leftrightarrow mL** button, then **ENTER**.
 7. Patient weight. Press **PROG**, then enter weight, then **ENTER**.
 8. Infusion Rate should be 10 $\mu\text{g}/\text{kg}/\text{min}$. **ENTER**.
 9. Occlusion Level should be High (1.2 bar). **ENTER**.
 10. Ready to run? Press **YES**.
 11. Infusion is ready, press **START/STOP** to start infusion. During infusion, a pre-alarm will sound, just silence it.
 12. To adjust dose during infusion, press **PROG**, type in new rate, then press **ENTER**.
- To silence alarm, press **C/button**.
 - To change something, press **PROG**, then adjust, then **ENTER**.
 - Can also refer to **Quick Reference Guide** for any more basic information. But understand that concentration at UW is 5 mg/mL, not 1 mg/ml, as in the guide.


Reviewed By:



Scott B. Perlman, MD, MS
Chief, Nuclear Medicine



Derek Fuerbringer, CNMT
Manager, Nuclear Medicine

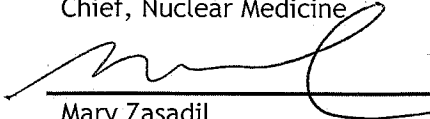

Mary Zasadil
Chief, Nuclear Cardiology



Scott Knishka, RPh, BCNP
Radiopharmacist