

THYROID UPTAKE PROBE - BioAssay Setup
UPDATED: MARCH 2010

CPT CODE: N/A

BIOASSAY SETUP

Investigational Level - The upper limit on the acceptable level of activity measured for each of three isotopes (I-131, I-125, and I-123). Measured activities above this level are flagged.

* Level is set at .01 uCi for I-131 *

Counting Time - Time per count for background and thyroid is set at 2 minutes.

Probe Distance - Probe distance is set at 24cm. Be sure to retract rod when counting.

Refer to Bioassay in Chapter 10 of CAPTUS 3000 manual (attached) for initial setup of system.

Reviewed By: S. Perlman, D. Fuerbringer

Scott B. Perlman, MD, MS
Chief, Nuclear Medicine

Derek Fuerbringer, CNMT
Manager, Nuclear Medicine